

UNIVERSAL  
**POWER  
BAR**<sup>®</sup>  
CLASS II APPLIANCE




## POWERBAR CLASS II CORRECTION FROM AMERICAN ORTHODONTICS

Manufactured at American Orthodontics' state-of-the-art facility in Sheboygan, WI, USA, PowerBar® is an aesthetic, fixed functional, direct bond orthodontic appliance used to bring Class II cases into Class I through efficient segmental tooth movement. The universal design of PowerBar eliminates the need for dedicated right and left appliances, allowing for placement on either side of the maxillary arch. Early application of PowerBar provides segmental correction, improving the efficiency of traditional or aligner treatments.




## EXPERIENCE THE POWERBAR CONSISTENCY

PowerBar is designed and engineered with the quality, convenience, and value you have come to expect from American Orthodontics. Using a proprietary process, each PowerBar is manufactured to maintain radius uniformity for treatment predictability. Like all AO products, PowerBar is engineered to comply with regulatory requirements, including UDI and MDR.

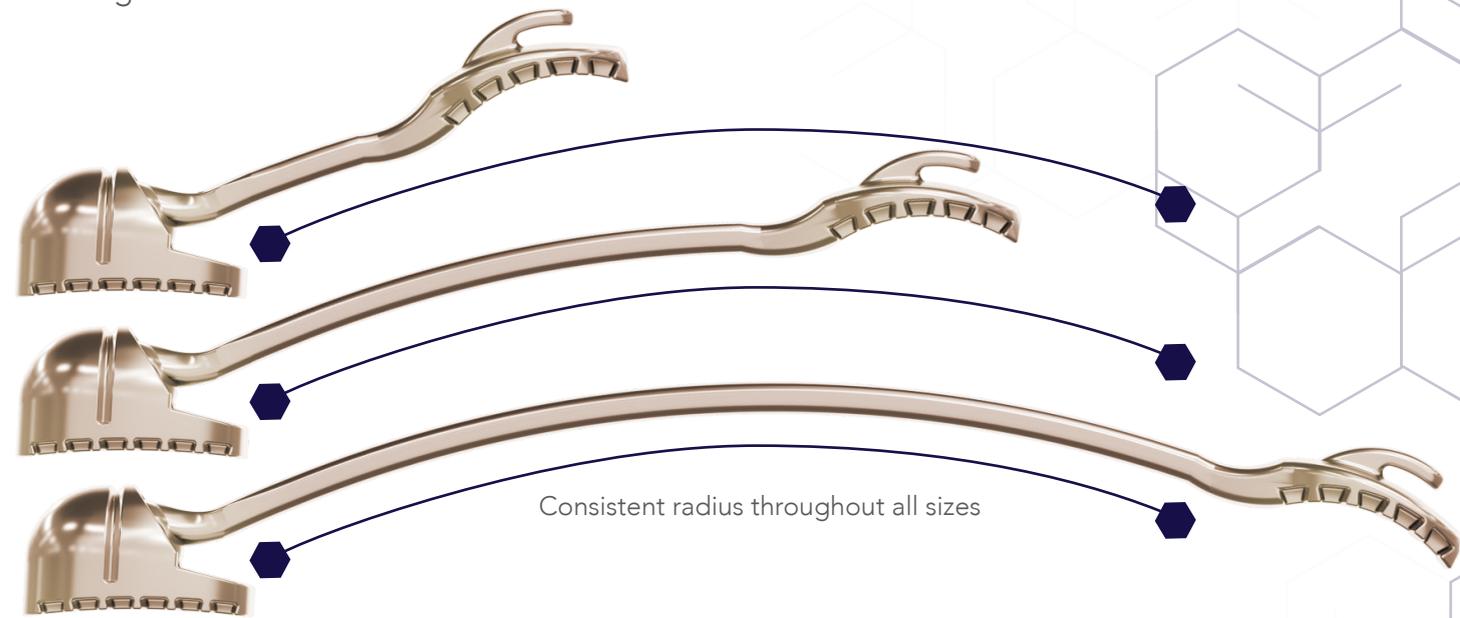

With a consistent elastic regimen, PowerBar is an early and efficient way to correct Class II malocclusions. PowerBar is designed to provide molar rotation and distalization during treatment, resulting in space between laterals and cuspids, or cuspids and premolars.

UNIVERSAL  
**POWER  
BAR**  
CLASS II APPLIANCE

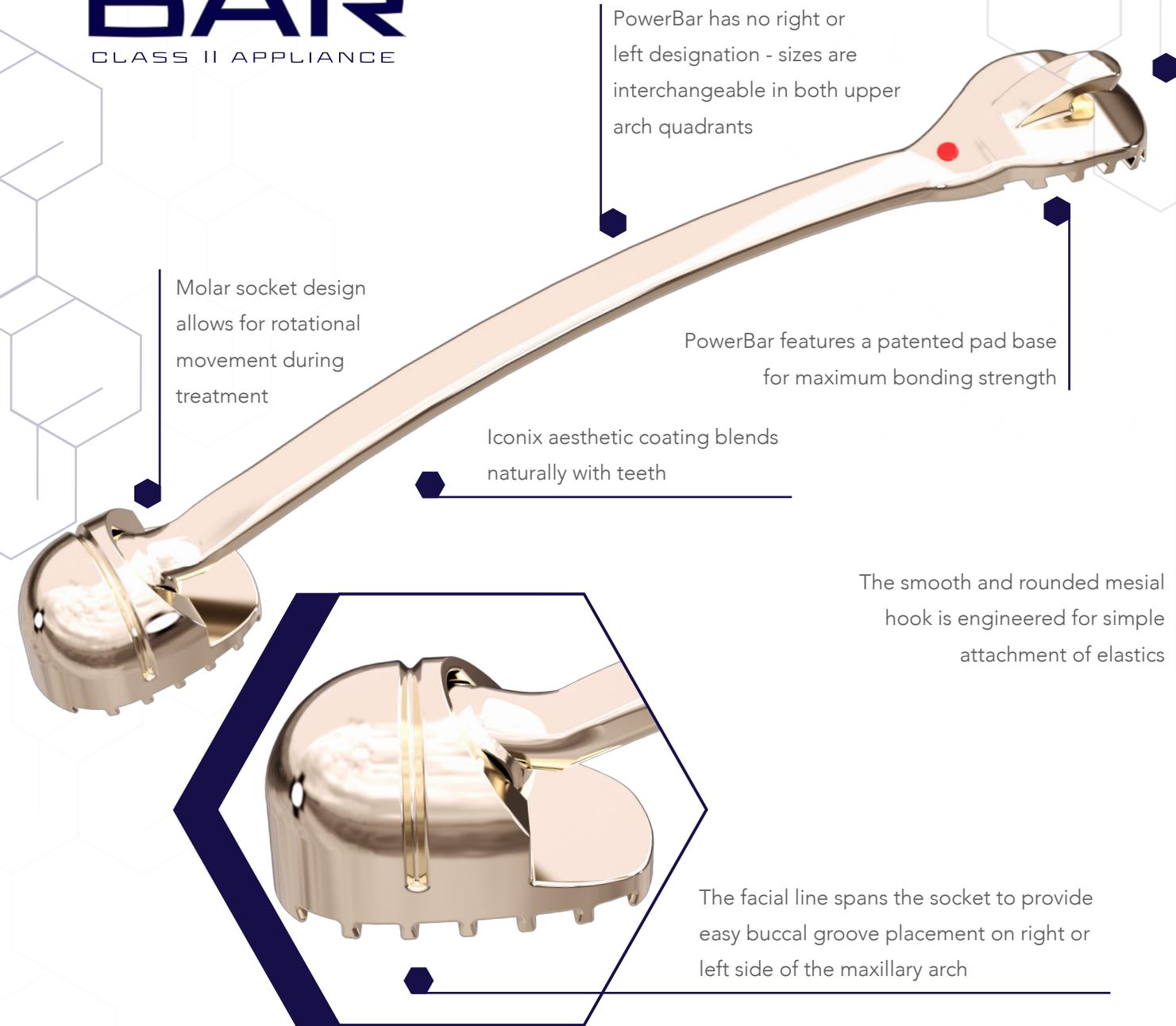

## SMART PACKAGING FOR INVENTORY CONTROL

PowerBar is available in 23 sizes, each individually color coded for visual inventory control. The American Orthodontics Smart Storage System is designed for quick identification and retrieval of appliances, and the universal design of PowerBar requires no dedicated right and left storage.



PowerBar packaging  
(2-1/4"W x 5/8"D x 4"H)



REF 909-037829  
18 wells, holds up to 72 PowerBar tubes  
(4"W x 8"D x 2"H)

## AVAILABLE IN 23 SIZES

The wide array of sizes provides you with the flexibility of spanning PowerBar from cuspid to molar, or pre-molar to molar in patients of all ages and growth levels.



## UNIVERSAL POWERBAR® APPLIANCE

2 per package

Part Number	Size	Color Code
855-012	12mm	Lime Green
855-013	13mm	Orange
855-014	14mm	Teal
855-015	15mm	Yellow
855-016	16mm	White
855-017	17mm	Red
855-018	18mm	Blue
855-019	19mm	Green
855-020	20mm	Pink
855-021	21mm	Purple
855-022	22mm	Black
855-023	23mm	Yellow
855-024	24mm	White
855-025	25mm	Red
855-026	26mm	Blue
855-027	27mm	Green
855-028	28mm	Pink
855-029	29mm	Purple
855-030	30mm	Black
855-031	31mm	Teal
855-032	32mm	Orange
855-033	33mm	Lime Green
855-034	34mm	No Color

LITFUNC-036747 Selection Guide 5 per package



## LET POWERBAR WORK FOR YOU


After scanning and anchoring the lower arch with a .040" plastic material like ATMOS and adding a buccal tube, button, or UT hook to the lower first or second molar, the PowerBar is placed on the upper arch.

Patients are typically given 6 ½ ounce, ¼ inch elastics, such as Sea Lion (non-latex is Marlin), to use during initial treatment. After the first month, 8 ounce, ⅜ inch elastics, such as Cheetah (non-latex is Stingray), are typically used. Once Class II is corrected, the PowerBar can be removed.

Talk to your AO Sales Representative and see how PowerBar can be incorporated into your practice.



Scan to see  
PowerBar in action

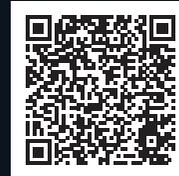



3524 Washington Avenue  
Sheboygan, Wisconsin, 53081 USA

Phone: +1 920 457 5051  
Email: [cs@americanortho.com](mailto:cs@americanortho.com)  
Web: [americanortho.com](http://americanortho.com)



©2024 American Orthodontics Corporation  
All Rights Reserved

FOLLOW US ON

